

FIV Ab/FelV Ag Combo Rapid Test (FIV-FelV)

Catalog No.: JCA104D

✧ INTENDED USE

The FIV Ab/FelV Ag Combo Rapid Test is a combined cassette to differentially diagnose the presence of Feline Immunodeficiency antibody and Feline Leukemia Virus antigen in cat's blood specimen.

Assay Time: 5-10 minutes

Specimen: Serum, plasma or whole blood

✧ PRINCIPLE

The FIV Ab/FelV Ag Combo Rapid Test is based on sandwich lateral flow immunochromatographic assay.

✧ REAGENTS AND MATERIALS

- Test devices
- Disposable capillary droppers
- Assay buffers for FIV and FeLV assay
- Products Manual

✧ STORAGE AND STABILITY

The kit can be stored at room temperature (4-30°C). The test kit is stable through the expiration date (24 months) marked on the package label. DO NOT FREEZE. Do not store the test kit in direct sunlight.


✧ SPECIMEN PREPARATION AND STORAGE

1. Specimen should be obtained and treated as below.
 - Serum or plasma: collect the whole blood for the patient cat, centrifuge it to get the serum, or place the whole blood into a tube which contains anticoagulants to get plasma.
 - Whole blood: collect fresh blood for use directly or make anticoagulant blood for storage at 2-8°C.
2. All specimen should be tested immediately. If not for testing right now, they should be stored at 2-8°C.

✧ TEST PROCEDURE

- Allow all materials, including specimen and test device, recover to 15-25°C before running the assay.
- Take out the test device from the foil pouch and place it horizontally.
- Using the capillary dropper to place 10 µL of the prepared specimen into the sample hole "S" of the test device. Then drop 3 drops (approx. 90 µL) of the assay buffer into the sample hole immediately.
- Interpret the result in 5-10 minutes. Result after 10

minutes is considered as invalid.

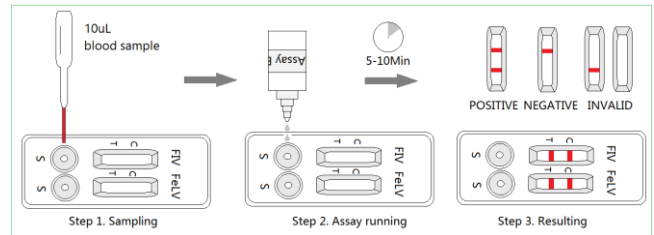

✧ INTERPRETATION OF RESULTS

- Positive (+): The presence of both "C" line and zone "T" line, no matter T line is clear or vague.
- Negative (-): Only clear C line appear. No T line.
- Invalid: No colored line appears in C zone. No matter if T line appears.

✧ PRECAUTIONS

- All reagents must be at room temperature before running the assay.
- Do not remove test cassette from its pouch until immediately before use.
- Do not use the test beyond its expiration date.
- The components in this kit have been quality control tested as standard batch unit. Do not mix components from different lot numbers.
- All specimens are of potential infection. It must be strictly treated according to the rules and regulations by local states.

✧ LIMITATION

The FIV Ab/FelV Ag Combo Rapid Test is for in vitro veterinary diagnosis use only. All result should be considered with other clinical information available with veterinarian. It is suggested to apply a further confirmative method such as Western Blot when positive result was observed.



J&G Biotech Ltd (Reg. No.: 08419172)

326 Cleveland Road, London, England E18 2AN, UK